HIT Control

Qualitative *HIT Control*, exclusively designed for the application in Milenia® QuickLine HIT

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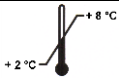
Note: Significant changes are indicated by dotted lines in the margin.
A change history can be found at the end of the manual.

Table of Contents

Explanation of Symbols	3
Warnings and Precautions	3
Intended Use	3
Materials supplied, Storage and Stability	4
Additional Materials Required (not included)	4
Volume Required	4
Preparation and Procedure	4
Interpretation of Results	5
Internal Quality Control Process	5
Additional Products available	5

Explanation of Symbols

Symbols	Explanation	Symbols	Explanation
	Use by date		Contains sufficient for <n> tests
	<i>In vitro</i> diagnostic medical device		Manufacturer
	Batch code		Consult Instructions for use
	Catalogue number		Control
	Temperature limitation		Positive Control
	Negative Control		

Warnings and Precautions

- Read this instruction manual and the instruction manual of Milenia® *QuickLine* HIT-Test (MQHIT 1/Z) carefully before performing the assay
- Follow the sequence of the work steps
- Aliquoted control should be thawed completely before usage
- Allow all reagents reach room temperature before use
- Handle Milenia® *QuickLine* HIT Control reagents carefully
- Wear lab coat and protective gloves
- The disposal of waste must be carried out according to local regulations
- Animal materials which are used in the reagent are free from infection pathogens and are derived from animals kept in Germany
- Human materials which are used in the reagent were tested and found negative for: HIV 1/2, HbsAg, HCV, HIV-1 (NAT) and RPR by FDA approved methods
- For professional users

Safety data sheets are available on request.

Intended Use

The Milenia®*QuickLine* HIT Control is used for internal quality control of the corresponding Milenia®*QuickLine* HIT-Test. The Milenia®*QuickLine* HIT Control kit contains a negative and a positive sample. It is used to determine test validity. The product is for in vitro diagnostic use.

Materials supplied, Storage and Stability

Component	Cat.-No.	Content	Preparation	Storage (unopened)	Shelf Life
QuickLine HIT Positive Control	MQHITC1	1 vial with lyophilisate, (50 tests)	lyophilisate; reconstitute with 0,25 mL distilled water not less than 30 min before usage	2°C to 8°C	lyophilised at 2°C to 8°C stable until the imprinted expiry date
QuickLine HIT Negative Control	MQHITC2	1 vial with lyophilisate, (50 tests)	lyophilisate; reconstitute with 0,25 mL distilled water not less than 30 min before usage	2°C to 8°C	lyophilised at 2°C to 8°C stable until the imprinted expiry date

Storage and component stability after reconstitution	Storage	Shelf Life
Control reconstituted in aqua dest.	2°C to 8°C	1 month
Control is aliquoted immediately after reconstitution	-20°C or below	3 month

Additional Materials Required (not included)

- Pipette and tips: 250 µL
- Distilled water
- Milenia® QuickLine HIT-Test (MQHIT 1 or MQHIT Z)
- Multi-axle rotating mixer

Volume Required

5 µL Milenia® QuickLine HIT Positive/Negative Control is required per test unit (MQHIT 1 or MQHIT Z).

Preparation and Procedure

a) Please follow these instructions to prepare the control material for first-time use

- Bring lyophilised Milenia® QuickLine HIT Positive and Negative Control to room temperature (18°C to 25°C) before dissolving
- **Add 250 µL distilled water** to each vial
- Close control vials
- Mix the Milenia® QuickLine HIT Positive and Negative Control thoroughly with e.g. a Vortex Mixer. Afterwards put the control vials for at least 30 minutes on a multi-axle rotating mixer (if possible).
- The Milenia® QuickLine HIT Control is now ready-to-use.
- For long-term storage (max. 3 month), it is recommended to divide, and aliquot Milenia® QuickLine HIT Control material and freeze at - 20 °C or below until it is needed. Do not refreeze.

b) Use of Milenia® QuickLine HIT Control:

- **Add 5 µL Milenia® QuickLine HIT Positive or Negative Control** in the MQHIT testunit. Before use, please consult instructions for use of *Milenia® QuickLine HIT* (Ref MQHIT 1/Z).
- Please note: The control material should be brought to room temperature, if it was stored frozen or cooled (at least 30 min on a multi-axle rotating mixer) before using it in the MQHIT Test.

Interpretation of Results

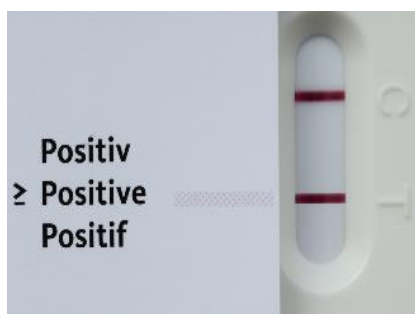
Fig.1: Positive control

- Milenia® QuickLine HIT **Positive** Control is considered positive, if the test line (T) shows the **same** or a **darker** shade of color than the given line on the Evaluation Card.

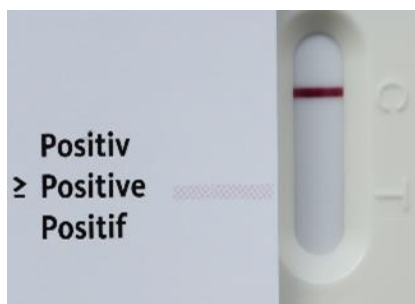
Fig.2: negative control

- Milenia® QuickLine HIT **Negative** Control is evaluated negative if the test line (T) does not appear or the line shows a **weaker shade of color** than the given line on the Evaluation Card.

Internal Quality Control Process

Each laboratory should establish quality control procedures that are tailored to their individual requirements and are in compliant with relevant regulations or the terms of approval, including corrective actions which must be taken if the results are outside of target ranges.

We recommend running a positive and a negative control when starting to use a new Lot.

In addition, the specifications written in the instructions of *Milenia® QuickLine HIT* tests (Ref MQHIT 1 or MQHIT Z) is subject to the evaluation of *Milenia® Quickline HIT Control*.

Additional Products available

Product Name	Order No.	Content	Description
Milenia® QuickLine HIT	MQHIT 1	20 tests	Qualitative detection of IgG antibodies against PF4/polyanion-complexes in human citrated plasma or serum
	MQHIT Z	5 tests	

For any questions and suggestions, please contact us:

Telefon: +49 641 – 94 88 83 – 0
Fax: +49 641 – 94 88 83 – 80
Web: www.milenia-biotec.de

HIT Control

Qualitative *HIT Control*, exclusively designed for the application in Milenia® *QuickLine* HIT

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